

Application Sheet

HSV IgM

REF 51126

Instrument: Elisys Duo

REF 17200

Setting:

HSVM_51126_R4.asy

For all essential product information, please refer to the User Manual of the analyzer and the instructions for use for reagents, controls, calibrators and the Certificate of Analysis. Please refer to the instruction for use for performance data.

This Application Sheet provides additional information regarding the use of the assay on the HUMAN ELISA analyzer.

The parameters defined in this application sheet have been developed to provide optimal product performance with the assay and instrument combination. Any modification to these parameters may negatively affect the performance and results of the assay. The user is responsible for the validation of any modification to the protocol here described.

Material Required

Material	REF	Comments
HSV IgM	51126	
Sample Rack Elisys Duo	17200/23	NC; PC; sample collection tubes
Reagent Rack (2 Lanes) Elisys Duo	17200/25	CON, SUB, STOP
Adaptor 100 ml Square Bottle	17200/30	DIL
Dilution Plate	17200/255	predilution of samples

Additional Notes

High OD-values of 3.5 and more are displayed as 4.0