

# Application Sheet

## Toxo IgG

**REF 51209**

## Instrument: Elisys Duo

**REF 17200**

### Setting:

**TOXOG\_51209\_R5.asy**

For all essential product information, please refer to the User Manual of the analyzer and the instructions for use for reagents, controls and calibrators. Please refer to the instruction for use for performance data.

This Application Sheet provides additional information regarding the use of the assay on the HUMAN ELISA analyzer.

The parameters defined in this application sheet have been developed to provide optimal product performance with the assay and instrument combination. Any modification to these parameters may negatively affect the performance and results of the assay. The user is responsible for the validation of any modification to the protocol here described.

### Material Required

Material	REF	Comments
Toxo IgG	51209	
Sample Rack Elisys Duo	17200/23	NC, PCs, CC sample collection tubes
Reagent Rack (2 Lanes) Elisys Duo	17200/25	CON, SUB, STOP
Adaptor 100 ml Square Bottle	17200/30	DIL
Dilution Plate	17200/255	predilution of samples

### Additional Notes

Due to a change of the Toxo IgG ELISA kit (REF 51209), the setting version R4 or lower should only be used up to LOT 20001. Please use setting version R5 or higher when using LOT 20002 or higher.

OD-values of 3.5 or above are outside the photometer's dynamic range. For this reason they are displayed as 4.0.