

Application Sheet

TSH

REF 54030

Instrument: Elisys Duo

REF 17200

Setting:

TSH_54030_R5

For all essential product information, please refer to the User Manual of the analyzer and the instructions for use for reagents, controls and calibrators. Please refer to the instruction for use for performance data.

This Application Sheet provides additional information regarding the use of the assay on the HUMAN ELISA analyzer.

The parameters defined in this application sheet have been developed to provide optimal product performance with the assay and instrument combination. Any modification to these parameters may negatively affect the performance and results of the assay. The user is responsible for the validation of any modification to the protocol here described.

Material Required

| Material | REF | Comments |
|-------------------------------------|----------|-------------------------|
| TSH | 54030 | |
| Sample Rack Elisys Duo | 17200/23 | sample collection tubes |
| Rack Controls Elisys Duo | 17200/24 | CAL A - F |
| Reagent Rack (2 Lanes) Elisys Duo | 17200/25 | CON, SUB, STOP |

Additional Notes

OD-values of 3.5 or above are outside the photometer's dynamic range. For this reason they are displayed as 4.0.