

Application Sheet

Toxo IgM μ -capture

REF 51119

Instrument: Elisys Duo

REF 17200

Setting:

TOXO_M_ μ C_51119_R4.asy

For all essential product information, please refer to the User Manual of the analyser and the Instructions for Use for reagents, controls and calibrators. Please refer to the Instructions for Use for performance data.

This Application Sheet provides additional information regarding the use of the assay on the HUMAN ELISA analyser.

The parameters defined in this application sheet have been developed to provide optimal product performance with the assay and instrument combination. Any modification to these parameters may negatively affect the performance and results of the assay. The user is responsible for the validation of any modification to the protocol here described.

Materials Required

Material	REF	Comments
Toxo IgM μ -capture	51119	
Sample Rack Elisys Duo	17200/23	NC, PCL, PCH sample collection tubes
Reagent Rack (2 Lanes) Elisys Duo	17200/25	CON, SUB, STOP
Adapter 100 ml Human Square Bottle	17200/30	DIL
Dilution Plate	17200/255	predilution of samples

Additional Notes

OD-values of 3.5 or above are outside the photometer's dynamic range. For this reason they are displayed as 4.0.