

Application Sheet

IMTEC-dsDNA-Antibodies

REF ITC59001

Instrument: Elisy Quattro

REF 16300

Setting:

dsDNA_ITC59001_R4.asy

For all essential product information, please refer to the User Manual of the analyzer and the instructions for use for reagents, controls, calibrators and the Certificate of Analysis. Please refer to the instructions for use for performance data.

This Application Sheet provides additional information regarding the use of the assay on the HUMAN ELISA analyzer.

The parameters defined in this application sheet have been developed to provide optimal product performance with the assay and instrument combination. Any modification to these parameters may negatively affect the performance and results of the assay. The user is responsible for the validation of any modification to the protocol here described.

Material Required

Material	REF	Comments
IMTEC-dsDNA-Antibodies	ITC59001	
Reagent Rack 0 Elisy Quattro	16300/38	CAL1 - CAL5, NC, PC, CON, SUB, STOP
Sample Rack 20 Elisy Quattro	16300/41	Suitable for sample collection tubes

Additional Notes

The concentration of the PC must be checked manually since the target values are batch specific.

OD-values of 3.5 or above are outside the photometer's dynamic range. For this reason they are displayed as 4.0.