

Application Sheet

Toxo IgM μ -capture

REF 51119

Instrument: Elisis Uno

REF 17350

Setting:

TOXO_M_ μ C_51119_R4

For all essential product information, please refer to the User Manual of the analyser and the Instructions for Use for reagents, controls and calibrators. Please refer to the Instructions for Use for performance data.

This Application Sheet provides additional information regarding the use of the assay on the HUMAN ELISA analyser.

The parameters defined in this application sheet have been developed to provide optimal product performance with the assay and instrument combination. Any modification to these parameters may negatively affect the performance and results of the assay. The user is responsible for the validation of any modification to the protocol here described.

Materials Required

Material	REF	Comments
Toxo IgM μ -capture	51119	
Reagent Rack Infect. Diseases & Steroids for Elisis Uno	17359/2	all reagents
Sample Rack, 96 Round Spaces 13 mm Elisis Uno, or 12 mm Elisis Uno	17359/8 17359/7	Samples and dilution, according to sample collection tubes
Sample vials 2 ml	17350/16	Sample dilution

Additional Notes

For technical reasons the assay step “start incubation time” requires to define a corresponding reagent. This reagent only exists in the software, there is no reagent bottle for it. However, this virtual reagent needs to be allocated to a position on the reagent rack, any position can be chosen. Do not place any other reagent bottle in this position while the virtual bottle is allocated to it.