

Application Sheet

PSA

REF 52030

Instrument: HumaReader HS

REF 16670

Setting:

PSA_52030_R5

For all essential product information, please refer to the User Manual of the analyzer and the instructions for use for reagents, controls and calibrators. Please refer to the instruction for use for performance data.

This Application Sheet provides additional information regarding the use of the assay on the HUMAN ELISA analyzer.

The parameters defined in this application sheet have been developed to provide optimal product performance with the assay and instrument combination. Any modification to these parameters may negatively affect the performance and results of the assay. The user is responsible for the validation of any modification to the protocol here described.

Additional Notes

The validation criterion for the duplicates of CAL F has to be checked manually. The difference between the duplicates should not exceed 10 %.

The validation criteria for the minimum OD of CAL F and the verification of the steady increase of the absorption values from CAL A to CAL F are automatically checked by HumaReader HS. If the criteria fail, no result will be displayed.