

# Application Sheet

## Syphilis Screen

**REF 51005**

## Instrument: HumaReader HS

**REF 16670**

### Setting:

**SYPHS\_51005\_R4**

For all essential product information, please refer to the User Manual of the analyzer and the instructions for use for reagents, controls, calibrators and the Certificate of Analysis. Please refer to the instruction for use for performance data.

This Application Sheet provides additional information regarding the use of the assay on the HUMAN ELISA analyzer.

The parameters defined in this application sheet have been developed to provide optimal product performance with the assay and instrument combination. Any modification to these parameters may negatively affect the performance and results of the assay. The user is responsible for the validation of any modification to the protocol here described.

### Additional Notes

If one NC is out of range the HumaReader does not perform an interpretation. If one NC only is invalid, then push "Exit" and delete the respective NC from the layout ("Clear"), start the measurement again to get a report based on two valid NC-values.